TÜRKÇE (TR)

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ENGLISH (EN)

For use with BioRad Variant™ II Turbo A1C

ELUTION BUFFER B

Always keep this package insert on hand and carefully read it before use.

INTENDED USE:

These products are intended for the determination of hemoglobin A1c (HbA1c) in human whole blood for use with VARIANT™ II TURBO series. For in vitro diagnostics use only.

SUMMARY AND PRINCIPLE:

The measurement of stable HbA1c is very useful in the diagnosis and treatment of diabetes and observations of diabetic conditions. The VARIANT™ II TURBO measures HbA1c in blood using reversed-phase cation exchange chromatography. Blood sample into several hemoglobin components based on high performance liquid chromatography (HPLC).

WARNINGS AND PRECAUTIONS**Warnings:**

- As this elution buffer contains an oxidizer, if elution buffer spills onto the countertop of floor, immediately and carefully wipe it up with a cloth, then rinse the collected liquid from the cloth with plenty of water. Then, rinse the collected liquid from the cloth with plenty of water.
- If the spilled liquid dries and crystallizes, wipe up the spill using a cloth moistened with water. Then, rinse the collected liquid from the cloth with plenty of water.
- Never contact spilled liquid or crystals with reducing agents such as alcohol or ascorbic acid.
- Dilute any remaining reagent in elution buffer packs with plenty of water before discarding.
- This elution buffer contains less than 0.01% mucosal- irritative sodium azide (NaN₃) as a preservative.

- Sodium azide may also react with copper or lead whereby potentially generating explosive metal azide.
- Sodium azide may also react with acid leading to the generation of toxic explosive hydrogen azide.
- Therefore, when these materials are disposed of, you should flush them with an adequate volume of water in order to prevent any residual reagent in the water servicepipe.
- If any of these elutions make contact with eyes or mouth, immediately wash with plenty of water and consult a doctor.
- If it makes contact with skin, wash with plenty of water.
- Discard liquid waste than contains blood sample in accordance with local regulations for biohazardous waste.

Precautions:

- For in vitro diagnostic use only.
- These products are only for VARIANT™ II TURBO series. DO NOT attempt to use with other instruments
- If these products are stored in a cold place, such as a refrigerator, allow them to adjust to the same environment as the instrument for at least one hour before placing them on the instrument.
- DO NOT use elution buffer packs beyond their expiration dates.
- The expiration dates are written on both the box and packlabel.
- Place each elution buffer pack in the specified position on the bottle tray.
- Attach the proper bottle cap with nozzle to each elution buffer B pack.
- Connecting the wrong bottle cap to the wrong pack may cause mixing of elution buffers, producing inaccurate measurement results.
- Replace the elution buffer pack with a new one even if a small volume of elution buffer remains.
- Adding new elution buffer to the old pack may cause inaccurate measurement results.
- Always handle these elution buffers with care.
- Avoid physical shock.

MEASUREMENT PROCEDURE

Refer to the operating manual for VARIANT™ II TURBO series.

STORAGE:

- Store unopened elution buffer packs at a temperature between 15°C and 30°C, avoiding direct sunlight.
- Once opened, use the elution buffers within one month, even if it is before expiration date.
- If the instrument is not to be used for one week or more, remove the elution buffer packs from the instrument. Then, cap them tightly and store them at a temperature between 15°C and 30°C in order to prevent liquid degradation and/or concentration.

MATERIALS PROVIDED:

ELUTION BUFFER B: 2000 mL X 1 pack/ box, succinate buffer

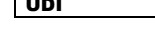
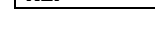
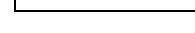
Materials of the pack:

Pack with PP (polypropylene) cap. The pack consists of polyethylene (PE) and polyethylene terephthalate (PET).

Archem Sağlık Sanayi ve Tic. A.Ş.
Mahmutbey Mah. Halkalı Cad. No:124 Kat:4
Bağcılar/İstanbul/Türkiye
Tel: +90 212 444 08 92
Fax: +90 212 629 98 89
info@archem.com.tr www.archem.com.tr

**SYMBOLS**

	In Vitro Diagnostic Medical Device
	Lot Number
	Unique Device Identifier
	Reference Number
	Identifies Products to Be Used Together
	Product of Turkey
	Manufacturer
	Expiration Date
	Temperature Limits
	Consult Instructions for Use